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Referências

1. Jiménez V. Mezclas intravenosas y nutrición artificial. Valencia: CONVASER; 1999. ISBN: 978-84-605-8427-8.
2. Katzung BG, editor. Basic & clinical pharmacology. Fourteenth edition. New York Chicago San Francisco Athens London Madrid Mexico City Milan New Delhi Singapore Sydney Toronto: McGraw-Hill Education; 2018. 1250 p. (A Lange medical book). ISBN: 978-1-259-64115-2.
3. REF-70862 Infusion Therapy Standards of Practice, 9th Edition. Journal of Infusion Nursing 44(1S):p S1-S224, January/February 2024. | DOI: 10.1097/NAN.0000000000000396
4. REF-70864 Moureau, N. L., Trick, N., Nifong, T. et al. Vessel health and preservation (part 1): A new evidence-based approach to vascular access selection and management. J Vasc Access. 2012;13(3):351-356.
5. REF-70866 Gallieni, M., Pittiruti, M. and Biffi, R. Vascular access in oncology patients. CA: a cancer journal for clinicians. 2008;58(6):323-346
6. REF-70865 https://www.health.qld.gov.au/_data/assets/pdf_file/0030/444486/icare-port-guideline.pdf
7. CS-004446 The BD Insyte Autoguard Shielded IV Catheter is designed to provide safety activation with a push of a button and is completely encapsulated after activation to help reduce the risk of needlestick injury.
8. CS-002575 BD Insyte™ Autoguard™ BC Pro Shielded IV Catheters with blood control technology demonstrated a 95% reduction in the risk of blood exposure.
9. BD Insyte Autoguard Shielded IV Catheter Instructions for Use.
10. CS-000147 González López J, Arribi Vilela A, Fernández Del Palacio E, et al. Indwell times, complications and costs of open vs closed safety peripheral intravenous catheters: a randomized study. J Hosp Infect. 2014;86(2):117-126.
11. González López J, Arribi Vilela A, Fernández Del Palacio E, et al. Indwell times, complications and costs of open vs closed safety peripheral intravenous catheters: a randomized study. J Hosp Infect. 2014;86(2):117-126.
12. PIVC Playbook.
13. CS-002643 Glass Particles contamination : Influence of aspiration and ampules types Sabon R.L.
14. CS-000162 Provena Midline IFU.
15. 0738901 PowerGlide Pro IFU.
16. En comparación con catéteres de punta no reforzada similares. Todos los datos están archivados en BD. Los resultados de las pruebas en el laboratorio pueden no ser predictivos del desempeño clínico real. BD-69539 Banner midline.
17. PowerPICC IFU.
18. REF-13094 0741196 PowerPICC Nursing Instructions for Use BD-72303424.
19. Lundberg PW, Smith AA, Heaney JB, et al. Pre-Operative Antisepsis Protocol Compliance and the Effect on Bacterial Load Reduction. Surg Infect (Larchmt). 2016;17(1):32-37.
20. Casey AL, Badia JM, Higgins A, et al. Skin antisepsis: it's not only what you use, it's the way that you use it. J Hosp Infect. 2017;96:221-222.
21. Hibbard JS. Analyses comparing the antimicrobial activity and safety of current antiseptic agents: a review. J Infus Nurs. 2005;28(3):194-207.
22. Haddadin Y, Annamaraju P, Regunath H. Central Line Associated Blood Stream Infections. StatPearls [Internet]. Treasure Island (FL): StatPearls Publishing; 2020 Jan.
23. (CS-004916): 60+ BD Chloraprep Clinical Studies categorized – 2023.
24. BD-89169 Site Rite 8 Brochure.
25. CS-003605 When compared to traditional PICC placement methods of blind placement and CXR, the Sherlock 3CG + TCS has demonstrated proven time and observed cost savings, specifically reduction in time to release PICC patients for IV therapy, malposition rates and chest X-rays.
26. CS-003602 The Sherlock 3CG+ TCS provides real-time navigation and confirmation of catheters during placement procedures. 51. Schears GJ. Summary of product trials for 10, 164 patients: comparing an intravenous stabilizing device to tape. J Infus. Nurs. 2006;29(4):225-31.
27. (CS-003736): Suture-Free Alternatives: New Approaches to Reduce Risk of Suture Needle Injuries – 2020.
28. CS-005376, CS-005378) Goossens et al, SecurAstaP trial: securement with SecurAcath versus StatLock for peripherally inserted central catheters, a randomised open trial, BMJ Open 2018.
29. (CS-005377): Goossens et al, SecurAstaP trial: securement with SecurAcath versus StatLock for peripherally inserted central catheters, a randomised open trial, BMJ Open 2018.
30. (CS-000282): PowerPICC Nursing Instructions for Use – 2020.
31. Data on File [C—0657, Claim 2: BD SmartSite™ – power injectable]. BD.
32. Data on File [C—2133, C-1270; Claim 6: BD SmartSite™ – reduce needlesticks]. BD.
33. Data on File [C—XXXX, Claim 3: BD Q-Syte™ – compatible with luer-slips]. BD.
34. Data on File [C-XXXX: BD Q-Syte™ – flow rate]. BD.
35. Rosenthal VD, Udawadia FE, Kumar S, et al. Clinical impact and cost effectiveness of split-septum and single-use prefilled flushing device vs 3-way stopcock on central-line associated bloodstream infection rates in India: a randomized clinical trial conducted by the International Nosocomial Infection Control Consortium (INICC). Am J Infect Control. 2015;43(10):1040-1045.
36. CDC. Guidelines for the prevention of intravascular catheter-related infections, 2011. CDC. Accessed May 5, 2020. <https://www.cdc.gov/hai/pdfs/bsi-guidelines-2011.pdf>
37. Williams A. Catheter occlusion in home infusion: The influence of needleless connector design on central catheter occlusion. J Infus Nurs. 2018;41(1):52-57. doi: 10.1097/NAN.0000000000000259.
38. Playbook Needle Free Connectors.
39. Williams A. Catheter occlusion in home infusion: The influence of needleless connector design on central catheter occlusion. J Infus Nurs. 2018;41(1):52-7.
40. Tabak YP, Johannes RS, Sun X, Crosby CT, Jarvis WR. Innovative use of existing public and private data sources for postmarketing surveillance of central line-associated bloodstream infections associated with intravenous needleless connectors. J Infus Nurs. 2014;39(5):328-35.
41. REF-16020 PosiFlush Technical Information.

Os logotipos BD e Família IAG™, PowerGlide Pro™, PowerPICC™, PowerPICC™ 3CG, BD PowerPICC® SOLO, BD PowerPICC® Radiology, BD Saf-T Intima™, BD Nexiva™, Provena™, Site-Rite™ 8, SherLock™ 3CG, BD StatLock™, BD SmartSite™, BD Q-Syte™, BD MaxZero™, BD PosiFlush™, BD Chloraprep™ são marcas registradas da Becton, Dickinson and Company. © 2025 BD. Todos os direitos reservados.